Air Flow Back Support
Ζώνη Οσφύος - Air Flow

EN | Instructions for Use

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EN

Air Flow Back Support

If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopedic consultant for advice.

FEATURES

This Air Flow belt is a light, breathable and extremely comfortable back brace that provides the basic support you need during normal day activities. Helps provide support to stiff, strained or aching back.

- Adjustable straps will help you find the ideal fit for you.
- Air-mesh fabric releases excess heat and moisture for all-day comfort.
- 4 Plastic stays provide firm grip, relief strain of lower back and protect weak muscles.
- Easy to care, dries quickly.

PERFORMANCE

The device provides lumbar spine gross motion restriction. The device reduces stress on the low back in flexion and extension.

INDICATIONS

Recommended for Lumbar Facet Syndrome, lower back pain, Sciatica, Osteoarthritis, moderate form of Spinal Muscular Atrophy, post-surgery back support.

WARNINGS . PRECAUTIONS OF USE

This product contains natural rubber latex which may cause allergic reactions. Not alternative for medical care. Seek professional medical advice for the diagnosis and treatment of pain, injury or irritation. with functional physiotherapy.

MEASUREMENT AND SIZE CHART

The size is to choose in accordance with the waist circumference, measured 2 cm above the iliac crests (in cm)

	S/M	L/XL
	70 - 100	90 - 125

SET UP AND OPERATION

- Release the straps, make sure they are both completely loose.
- Place the belt around your lower back and fasten the hook and loop fasteners.
- Pull the straps handles to reach to desired compression, then fasten and secure them on your belt (as shown in the picture).

CARE INSTRUCTION

Hand-wash with lukewarm water and gentle detergent. Hang and leave to dry.

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