MC 11

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If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopedic consultant for advice.

FEATURES

- Simple construction, easy to put on and take off
- Discreet fit underneath clothing
- Can be used as a waist trimmer
- Made of lightweight Neoprene that retains heat, with perforated parts to improve breathability and sweat management during activity
- Hook & Loop fasteners, allow lots of adjustability for customized fit
- Minimal, clean design and highly durable

PERFORMANCE

This belt provides mild compression and support for weak and strained abdominal muscles resulting from injury, exhaustion and inactivity.

INDICATIONS

Recommended for:

- Post-surgical or post-natal management, including Liposuction, Cesarean or Hysterectomy
- Relieving pain and strain in abdominal muscles
- Improving circulation and encouraging quicker recovery of abdominal injuries or post-surgery

WARNINGS. PRECAUTIONS OF USE

Do not use the product if you are allergic to any of its components. This product is not an alternative to medical care. Seek professional medical advice for the diagnosis and treatment of pain, injury or irritation.

SET UP AND OPERATION

- Place the belt around your waist (as shown in the picture).
- Position in the desired height.
- Adjust the hook & loop fasteners to fit tightly yet comfortably around your abdomen/lower back.

CARE INSTRUCTION

- Make sure to secure all hook and loop fasteners before washing, so they don't stick to other fabrics.
- Machine wash at low-temperature and gentle spin or hand-wash with lukewarm water and soft detergent.
- Do not tumble dry, dry clean, use bleach or fabric softeners.

MEASUREMENT AND SIZE CHART

The size is to choose in accordance with the waist circumference, measured 2 cm above the iliac crests (in cm)

	S/M	L/XL
	75 - 95	95 - 115

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